

Adjustable cough-relief valve

- Less pressure on your airways.
- Reduces pressure on the airtight tracheostoma seal.
- Closes automatically after you have coughed.

Simple to operate

- Easy to keep the valve closed.
- Designed to be less tiring to use.
- Makes hands-free speech available to more people.

Side-opening benefits

- Easy to cover with clothing.
- Body heat warms and moisturizes the air.
- No direct inflow of cold air.
- Prevents wet clothing.

Heat and Moisture Exchanger (HME) located under the valve

- Reduces mucus production.
- Reduces coughing.
- Better lung protection.
- Protects the valve against secretions.

Attachment options

- Different base plates for different skin types / stoma shapes.
- Extra glue as needed.
- LaryTube for use with a shrinking stoma.

Order information

Product	REF
Provox® FreeHands HME (whole set) (1pc), contents listed below:	7710
1 Valve unit, incl. a medium (blue dot) Speech Valve Membrane	
5 FreeHands HME Cassettes	
2 FlexiDerm Oval Adhesive base plates	
1 Adjustment Kit	
1 Speech Valve Membrane, light (white dot)	
1 Speech Valve Membrane, strong (green dot)	
1 Cleaning and Storage Box	
5 Skin-Prep™ tissues	
1 Instruction for Use	
Provox® FreeHands HME Cassette (20 pcs)	7712
Provox® FreeHands Speech Valve Membrane, light (white dot) (1 pc, includes 1 pc 7719)	7713
Provox® FreeHands Speech Valve Membrane, medium (blue dot) (1 pc, includes 1 pc 7719)	7714
Provox® FreeHands Speech Valve Membrane, strong (green dot) (1 pc, includes 1 pc 7719)	7715
Provox® FreeHands Cleaning and Storage Box (1 pc)	7718
Provox® FreeHands HME Adjustment Kit (1 pc safety screwdriver and 1 pc membrane replacement forceps)	7719
Regular Round Adhesive (20 pcs)	7251
Regular Oval Adhesive (20 pcs)	7252
FlexiDerm Round Adhesive (20 pcs)	7253
FlexiDerm Oval Adhesive (20 pcs)	7254
OptiDerm Round Adhesive (20 pcs)	7255
OptiDerm Oval Adhesive (20 pcs)	7256
LaryTube with Ring (incl. 5 pcs HiFlow and 1 pc FlexiDerm Oval)	
Size 8/36	7624
Size 8/55	7625
Size 9/36	7626
Size 9/55	7627
Size 10/36	7628
Size 10/55	7629
Size 12/36	7630
Size 12/55	7631
Provox® Silicone Glue (1 container, 40 ml)	7720
Provox® Cleaning Towel (200 pcs)	7244
Remove (50 pcs)	59403125
Remove (50 pcs) U.S. Version	403100
SkinPrep™ (50 pcs)	59420425
SkinPrep™ (50 pcs) U.S. Version	420400



© Atos Medical AB, 200105, REF 7982.
 Provox is a registered trademark owned by Atos Medical AB, Sweden. Skin-Prep and Remove are trademarks of Smith & Nephew United, Inc., USA.
 Provox HME is protected by U.S. patent (5,738,095) and other patents and patents pending. Provox FreeHands HME has a patent pending.
 U.S. federal law restricts this device to sale by or on order of a physician or licensed practitioner trained in post laryngectomy vocal and pulmonary rehabilitation procedures.

Pennan 01.50'10

PROVOX® FreeHands HME™

Automatic Tracheostoma Valve System



...for a more normal life after a total laryngectomy

Cassette (has REF 7240 or 7241 and is different from the FreeHands HME Cassettes!). Remember, an HME must be worn both day and night to be most effective – like your nose!

How long will this device last, and can I get spare parts?

The HME cassette is disposable and has to be replaced at least once a day, so it will work properly and to minimize the risk of contamination.

The valve unit can be cleaned and re-used as long as it works and shows no signs of cracks or other damage. You may have to replace the speech valve membrane after it has been used for some time. All accessories are available as spare parts.

What about cleaning and maintenance?

Place the reusable valve unit in the cleaning container for about 15 minutes, together with some water and an ordinary denture-cleaning tablet. Rinse it thoroughly under running water and let it dry in the air overnight. It's that simple.

ATOS MEDICAL

Atos Medical AB
 P.O. Box 183, SE-242 22 Hörby, Sweden
 Tel Int. + 46 415 198 00 • Fax Int. +46 415 198 98
 info@atosmedical.com • www.atosmedical.com

CE 0413



Read instructions before use

US Inquiries:
 Atos Medical Inc.
 2202 N Bartlett Ave., Milwaukee, WI 53202, USA
 Tollfree: 1-800-217 0025 • Fax: 1-414-227 9033
 E-mail: info.us@atosmedical.com
 U.S. Webshop: www.atosdirect.com

ATOS MEDICAL

A Perbio Science Company